


Risk Assessment

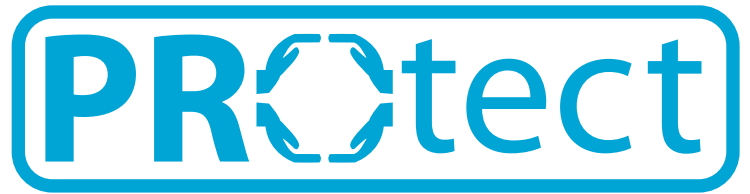

INDICATIONS FOR USE:

- Patients considered to be at risk of line pulling or disrupting lifesaving treatment via tube pulling
- Patients with a skin condition, whose scratching, exacerbates the condition
- Patients whose scratching could cause self harm or compromise the integrity of the wound site
- Psychiatric patients for whom limited movement would be beneficial for safety purposes
- Post hand surgery patients whose hands require cushioned protection

CONTRAINDICATIONS:

- DO NOT USE if patient IV site or wound site could be compromised by the mitt
- DO NOT USE if affected limb is fractured or dislocated
- DO NOT USE if a patient is highly aggressive, agitated or suicidal

IMPORTANT:

- If the applied device severely limits movement or is uncomfortable, this could lead to severe emotional, psychological and physical problems. If such symptoms appear or worsen it is recommended to reassess use of this product for this patient
- Ensure mitts are properly secured
- Ensure mitts are not torn or damaged
- Check patient cannot use his/her teeth to remove mitts
- Do not allow patient to ingest mitt material
- Be aware patients wearing mitts whilst walking could be at risk of falling and should be closely monitored

TalarMade Ltd
Springwood House, Foxwood Way
Foxwood Industrial Estate
Chesterfield, S41 9RN, UK
Tel: 01246 268456
Fax: 01246 268648
Web: www.TalarMade.com
Email enquiries: info@talarmade.com
Email orders: orders@talarmade.com