


DANIELS HEALTHCARE LTD SHARPSGUARD® Danicentre mounting instructions

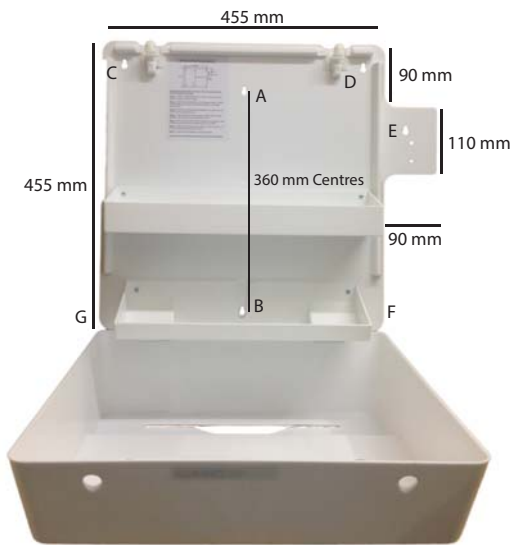

Please use the correct type of raw plug or fixing to suit the wall onto which the cabinet is to be fixed

Stage 1 - Select an unobstructed area on a wall measuring 540mm in length x 475mm in height.

Stage 2 - Mark the top centre hole (A) at a convenient height (1720mm recommended) and measure down vertically 360mm to the bottom centre hole (B).

Stage 3 - Fix 2 off round head screws (No8x25mm) into each hole and screw in until they are 6mm proud.


Stage 4 - Hang cabinets onto fixing screws by means of the 2 centre keyholes. Mark the remaining 5 hole positions (C-G).

Stage 5 - Fit the mounting bolts onto the recesses on the extension plate on the right hand side, if the hand dispenser unit is to be fitted.

Stage 6 - Security fix the back plate by means of screws into holes C,D,E,F and G and fully tighten screws A and B.

Stage 7 - Mount the hand dispenser unit and fully tighten.

Daniels accepts no responsibility for damages if screws are incorrectly fitted, or all screws are not used.

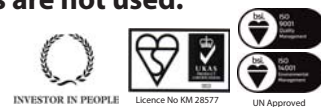

Danicentre_Mounting Instructions_v2



Download app
for 24/7 Support



A MAUSER Company

Safety, Service and Support - Every Day, Affordably & Responsibly

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.

The Application of Innovation

www.daniels.co.uk © DANIELS HEALTHCARE LTD 2017