


Setup Instructions

Read before using your Replacement Feet

Replacement Feet

CAUTION: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Using a Phillips screwdriver, remove screw (by turning counterclockwise) from center of each of the three (3) HurryCane® feet. Then remove rubber feet and discard the old feet and screws.

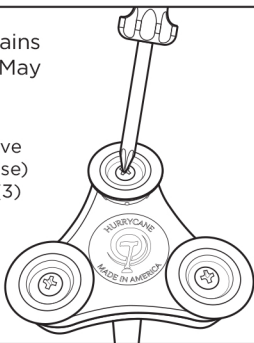

1

Carefully place new rubber foot onto pivoting head foot cavity.

Slide metal washer onto screw. Place screw with washer through rubber foot and into the threaded hole of pivoting head foot cavity. Be sure to tighten screw securely.

Repeat this step for the remaining two feet. HurryCane requires the feet to be replaced every six months. New feet can be purchased at www.HurryCane.com.


Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136

E-mail: success@hurrycane.com

HurryCane.com

Like us on Facebook: [facebook.com/AllTerrainCane](https://www.facebook.com/AllTerrainCane)

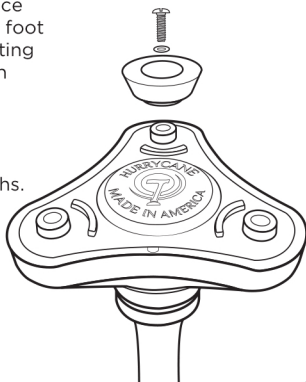

2



HURRYCANE®

The all-terrain cane