




# User Manual

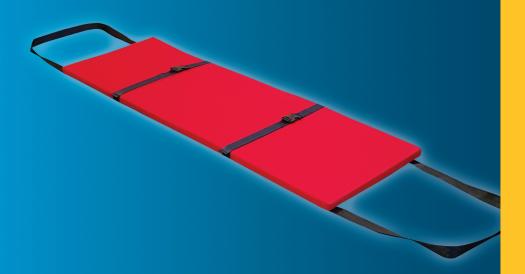

ALT-E002 Alerta Evacuation Sledge



The ALT-E002 Alerta Evacuation Sledge is wall mounted and ideally positioned at a nurse station, stored in an easily accessible wall mounted bag.

## Set Up

In the event of an evacuation pull the sledge from the storage bag, unfold and lay out on the floor. Position the user on the sledge and secure using the chest and leg straps. Be careful not to tighten the straps excessively.

### Evacuation

It is recommended to have a minimum of 2 people for performing evacuation, one at each end of the evacuation sledge. Holding the pulling straps firmly, carry or slide the user along the floor to safety. Take care to avoid sharp objects that may pierce the fabric.

### Maintenance & Warranty

The Alerta Evacuation Sledge is a low maintenance product. Regular inspections are recommended to ensure that the product remains intact. This is a single use product and warranty is void after use in evacuation.

# Cleaning & Care

Wipe clean using diluted chlorine based solution. Do not use bleach or biological cleaner.



# Making cutting-edge medical equipment affordable

### **Head Office**

Alerta Medical 4 Symington Place Riverside Business Park Irvine KA11 5DE United Kingdom

