

FLEXIBLE BUNION CORRECTOR

INDICATIONS

Wearing the EPITACT® flexible bunion corrector helps to correct the deformation and limits the development of the bunion.

CONTRA-INDICATIONS


The corrector should not be used if the big toe can no longer move laterally. In this case, choose the EPITACT® bunion protector. Do not use on injured skin. If you suffer from arteritis, diabetes, lower limb neuropathy or ulcers, seek advice from your doctor or podiatrist before use and then regularly monitor your foot.

PRECAUTIONS FOR USE

First check that the great toe can still move laterally when manipulated. If it cannot, do not use this device. Do not use this product overnight. The corrector can be used on either the right or the left foot.

TIPS FOR USE

Insert the forefoot in the strap, then put a hood on the great toe. Place the Epithelium™ patch (a) behind the bunion (b) (See Diagram 1). On the first day, do not try to straighten the great toe. Over 10 days, progressively straighten the great toe by pulling the device backwards (See Diagram 2). The device should be worn daily for maximum efficiency.

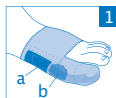

FEATURES

Thanks to its patented design with EPITHELIUMFLEX® technology, the ultra-thin and lightweight corrector is able to fit comfortably in any shoe.

CLEANING & LIFESPAN

Wash in a machine at 30°C / 86°F or by hand with soap and water (without rubbing) and leave to dry naturally. Do not iron. Reusable for several months.

COMPOSITION: 58% silicone, 42% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2011.



Machine washable at 30°C / 86°F.



Do not iron or steam.



Do not dry clean.



Do not bleach.



Do not tumble-dry.



See packaging.

Product of MILLET Innovation research.



MILLET Innovation ZA Champgrand
26270 Loriol/Drôme - FRANCE

MILLET
INNOVATION

www.epitact.co.uk

N10448EN-I
09/2018