

## RIGID THUMB BRACE

### HOLDS THE THUMB IN A RESTING POSITION



#### INDICATIONS:

During the night or in any situation in which you are not using your hand, regularly wearing the EPITACT® rigid thumb brace for rest helps relieve joint pains at the base of the thumb and helps restrict development of rhizarthrosis.

#### CONTRA-INDICATIONS:

Not for use on cutaneous lesions.

#### PRECAUTIONS FOR USE:

In case of severe oedema: monitor the hand during use. Remove the device if excessively tight and hampering circulation. If in doubt, ask your doctor's or pharmacist's advice.

Do not apply any cream or ointment before putting on the orthosis.

Do not use near a flame or incandescent material.

#### FEATURES:

The EPITACT® brace immobilises the thumb in resting position, opening the commissure between the index finger and thumb. The alignment of the bones limits the forces applied to the trapeziometacarpal joint and relieves pain.

#### ERGONOMICS:

The rigid thumb brace offers you optimum comfort owing to its reinforcements modelled to adapt perfectly to the morphology of your hand. Light and extra thin, you will forget you are wearing it.

#### TIP:

To supplement the rigid thumb brace, we recommend that you wear the EPITACT® flexible activity orthosis in all situations in which you need to use your hand.

#### LIFESPAN:

Hand washing at 30°C / 86 F, with a mild soap, without detergent or bleach. Rinse well. Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers.

**COMPOSITION:** 57% textile (Polyamide, Elasthan, Polyester), 43% TPE thermoplastic elastomer.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2017.



Hand washable.



Do not iron or steam.



Do not dry clean.



Do not bleach.



Do not tumble-dry.



See packaging.

Product of MILLET Innovation research.



MILLET Innovation ZA Champgrand  
26270 Lorio/Drôme - FRANCE

MILLET  
INNOVATION

www.epitact.co.uk

N19455EN-A  
09/2018