➤ HEINE Handles

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
MED 112679 2016-06-30

HEINE Handles

DEUTSCH	4
ENGLISH	8
FRANÇAIS	12
ESPAÑOL	16
ITALIANO	20
SVENSKA	24
NEDERLANDS	28
DANSK	32
NORSK	36
SUOMI	40
PORTUGUÊS	44

HEINE handle variants

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HEINE Handles

This instruction document is valid for the following products from the HEINE handle series: BETA battery handle, rechargeable handle: BETA NT, BETA 4 NT, BETA TR, BETA R, BETA 4 USB, BETA L, large battery handle, mini3000 battery handle and mini3000 rechargeable handle

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The HEINE handles are designed exclusively for the operational use of medical examination instruments manufactured by the HEINE company.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

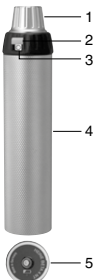
NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Product overview

- 1 AV-/TL-Connector
- 2 Unlock button
- 3 Rheostat
- 4 Outer sleeve
- 5 Bottom insert
- 6 Switch
- 7 Clip

BETA handles

mini3000 handles

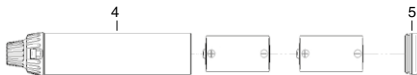
Setting up

Inserting battery and rechargeable battery

HEINE battery handle

To set up the HEINE battery handle, unscrew the bottom insert (5) from the handle module and insert the batteries with the positive terminal first into the outer sleeve (4). Screw the bottom insert (5) back in.

HEINE rechargeable handle

To set up the HEINE rechargeable handle, unscrew the bottom insert (5) from the handle module and insert the rechargeable battery into the outer sleeve (4) as described in the rechargeable battery instructions. Screw the bottom insert back in.

With the BETA 4 NT and BETA 4 USB handles, remove the insulation film between the bottom insert and the battery.

Changing of the Li-ion L battery

For BETA L, BETA 4 NT or BETA 4 USB, remove the bottom inserts as described above. Remove the battery from the bottom insert by pulling it away from the bottom insert and replace it with a new battery. Battery and bottom insert are connected correctly once a distinct click sound has been heard. Screw the bottom insert with the battery back into the handle.

Only charge the rechargeable handle with the following HEINE chargers:

NT300: BETA NT and BETA L rechargeable handle

miniNT: mini3000 rechargeable handle

NT4: BETA 4 NT rechargeable handle

E4-USB-power adapter: BETA 4 USB rechargeable handle

Setup and operation of the HEINE chargers are described in separate instruction documents.

Charging of the Li-ion L battery

The BETA L, BETA 4 NT and BETA 4 USB handles are designed specifically for the use of the Li-ion L rechargeable battery. To check the battery charge, the bottom inserts are equipped with a charge level indicator.

- BETA L: The charge level indicator is automatically activated when the instrument is disconnected from the charging device. With the BETA L, the display activates when it is turned on and goes off automatically after 10 seconds.

Charge status indicators:

Orange/green/green: 66 – 100 %

Orange/green: 33 – 66 %

Orange: 10 – 33 %

Orange flashing: < 10 %

- BETA 4 NT: The charge level indicator shows a constant orange when the battery needs to be recharged. With a fully discharged battery, the charge level indicator goes out and the instrument head is disabled.

- BETA 4 USB: While charging, the charge level indicator flashes green. After charging is complete, the battery indicator turns constant green. The charge level indicator shows a constant orange when the battery needs charging. With a fully discharged battery, the battery level indicator goes out and the instrument head is disabled.

Attaching the instrument heads

HEINE BETA® handle and large battery handle

Automatic Connector (AV)

Attach the instrument head onto the handle from above in such a way that the connector fits into the guides on the handle head. Push the instrument into the handle head until it locks in place with an audible "click".

To remove the instrument head turn the Connector (1) on the handle head to the right with the thumb and forefinger and, using the other hand, pull the instrument head up and out.

Twist-Lock Connector (TL)

To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

Removing the instrument

To remove the head rotate it anti-clockwise and pull it off.

HEINE mini3000® handle

The instrument head can be screwed on and off using the internal thread in the handle head.

Operation

HEINE BETA® handles and large battery handle

To activate the instruments press the unlock button (2) and turn the rheostat (3) clockwise. The light intensity is indicated by the bar .

Position "1" shows the maximum brightness; you can set any intermediate level as desired. To turn the device off, turn the rheostat back to position „0“.

HEINE mini3000® handles

To turn the instrument on, push the switch (6) down; to turn it off, push it back up.

When the clip (7) is clipped onto a breast pocket the handle turns off automatically.

HEINE BETA® handle variants

The handle module (outer sleeve and handle head) is the same for all HEINE BETA® handles (with the exception of the large battery handle). The handle type is determined by the bottom insert and the battery type. Deployment of a different HEINE BETA® handle is easily achieved by exchanging the bottom insert and/or battery type. An overview of the possible variants is shown on page 3.

Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: noncritical

Spaulding Classification USA: noncritical

 Allow the device to cool down before reprocessing.

Before cleaning, disconnect the device from the power source.

HEINE mini3000®: Remove this prior to processing the rechargeable battery.

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the cleansing agents and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues, automated or ultrasonic reprocessing.

 Ensure that the instrument is completely dry after reprocessing before you use it again.

Procedure

Clean and disinfect the HEINE handles manually (wipe clean and wipe disinfect).

Recommended agents

Cleaning agent: Neodisher® MediClean

Disinfectant agent BETA handles: alcoholic (e.g. Incides® Tissues) or quaternary ammonia compounds (e.g. Microbac® tissues)

Disinfectant agent mini3000 handles: quaternary ammonia compounds (e.g. Microbac® tissues)

Maintenance and Service

The instruments do not require maintenance or service.

General Warnings

 Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the safety requirements from the IEC 60950-1 Information technology or IEC 60601-1 Medical electrical equipment.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Turn the handle off during the charging process.

The handle is not designed for continuous use. The operating time of the handle depends on the maximum operating time of the examination head.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

If you don't use the device for a longer period of time, please remove the batteries in advance.

Ensure that moisture does not penetrate the area around the rechargeable battery and the bottom cap. Check the rechargeable batteries regularly. In case of any abnormalities in the operating or recharging times, replace the rechargeable batteries.

If the rechargeable battery has not been recharged for a long period it may discharge automatically.

Precautions for HEINE rechargeable batteries

Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.

Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage).

The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Dispose at your local collection point.

The appendix contains following tables

- Technical specification
- Technical specification HEINE BETA® R, BETA TR and BETA 4 USB Rechargeable handles
- Technical specifications of the HEINE rechargeable batteries
- Explanation of the used symbols.

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Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Recommended charging temperature for the rechargeable batteries	+10 °C to +35 °C

Technical specification

BETA R, BETA TR, 4 USB rechargeable handles

	BETA R	BETA TR	BETA 4 USB
Nominal voltage	230 V / 50-60 Hz	100-240 V / 50-60 Hz	5 V DC
Typical power and current	max. 2 VA	max. 9 VA	max. 500 mA
Protection class	Charging: II		Charging*: I / II
	In use: internal power supply		
Charging current	approx. 75 mA	approx. 75 mA	approx. 500 mA

*) dependant on the USB power supply

Technical specification of the HEINE rechargeable batteries

	HEINE Li-ion L (X-007.99.383)	HEINE Li-ion (X-007.99.381)	HEINE NiMH (X-002.99.382)	HEINE mini2Z (X-001.99.487)
Cell type	Lithium-ion technology		Nickel-metal hybrid	
Rated voltage	3.7 V		3.6 V	2.4 V
Capacity	2300 mAh		1000 mAh	750 mAh
Charging time NT300/miniNT	approx. 2 h	approx. 4 h		
Charging time NT4	approx. 2 h	-		
Charging time USB-power adaptor	approx. 4 h	-		
Dimensions	Ø 26 x 83 mm	Ø 26 x 82 mm	Ø 26 x 82 mm	Ø 14.4 x 107 mm
Weight	62 g	68 g	70 g	37 g

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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