
# Molift Raiser Pro User manual

a part of Etac

BM43099 Rev. B 2018-11-22

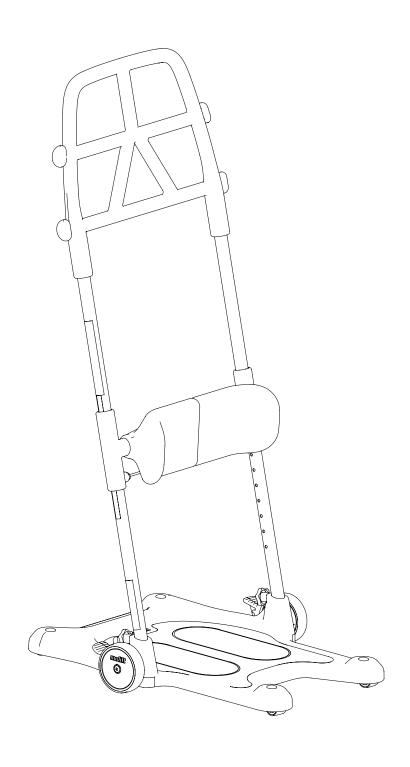



# Contents

- User manual English 4
- Bruksanvisning Svenska 8
  - Brukermanual Norsk 12
- Brugsvejledning Dansk 16
  - Käyttöohje Suomi 20
- Gebrauchsanweisung Deutsch 24
  - Handleiding Nederlands 28
  - Manual d'utilisation Français 32
    - Manual utente Italiano 36
  - Manual de usuarioo Español 40

#### Genera

# $\epsilon$

#### Declaration of conformity

The Molift Raiser Pro described in this user manual is CE marked in accordance with EU Council Directive 93/42/EEC concerning medical devices, class 1, and has been tested according to standard SS-EN12182:2012

#### Conditions for Use:

Transferring a person may pose an element of risk. Only trained personnel should use the equipment covered by this user manual.

#### Warranty:

2-year warranty against defects in workmanship and materials of the Product. For Terms and conditions, see www.etac.com

Visit www.etac.com to download the latest version of our doccumentation.

#### Important

#### Read User Manual before use!

It is important to fully understand the content of the user manual before attempting to use the equipment. This User Manual contains important safety instructions and information regarding the use of the Molift Raiser Pro. For instructions/information regarding the accessories, see separate user manual.

In this manual the user is the person being transferred. The assistant is the person operating the Molift Raiser Pro. Modifications and use of components made by other manufacturers:

We recommend only using Etac components and spare parts. Declaration of conformity is not valid and Etac is not responsible for warranty if any modifications are made to the Product. Etac shall not be liable for faults or accidents that can occur when using components made by other manufacturers.

#### Risk assessment

It is important that the Molift Raiser Pro has been tested with the individual user and for the intended transfer. Assess the risks and take notes. Evaluate the user's ability to use the Product and if needed use any suitable Molift Raiser Pro accessories. You as a carer are responsible for the safety of the user.



Only technical personnel are allowed to perform repairs.

#### Molift Raiser Pro - intended use

Molift Raiser Pro is a sit-to-stand transfer aid used indoors on flat surfaces for shorter transfer from one sitting position to another sitting position. Transfer between wheelchair and bed/toilet or chair. Can also be used for repositioning farther back in the wheelchair. Users must be active, able to grip and pull up their own body weight. Raiser Pro is designed for easy maneuvering and works well in confined spaces.

#### Labels and symbols

#### Product label:

The label contains art. number and serial number (bar codes). The EAN code contains;

- (01) (EAN no.): xxxxxxxxxxxxxx
- (11) (Batch): YYMMDD
- (21) (Serial number): xxxxx



#### Symbols:



CE marked



Refer to user manual



Max temperature/time when washing.



Max user weight (SWL)



pH when washing




Important safety information

#### Short user guide:





## Molift Raiser Pro

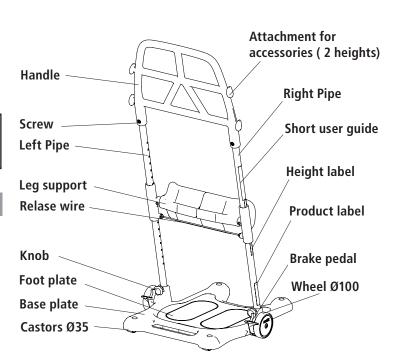

# Technical data

# Safe Working Load (SWL):

170kg (375 lbs)

#### Weight of unit:

10kg

#### Dimensions:


Height (A): 1,2 m
Height (B): 0,3-0,51 m
Depth (C): 0,63 m
Width (D): 0,54 m
Width (E): 0,37m
Step in height (F): 0,035m

#### Material:


Powder coated aluminum, Plastic, Steel.

#### Expected Lifetime:


The Molift Raiser Pro has an expected lifetime of 5 years or 15000 transport cycles.

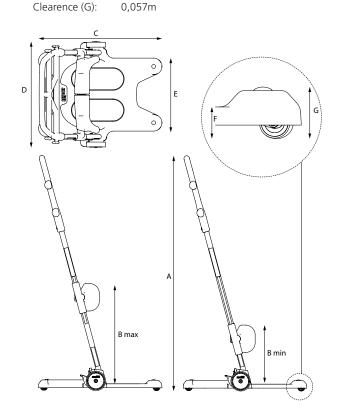



# Supplied parts

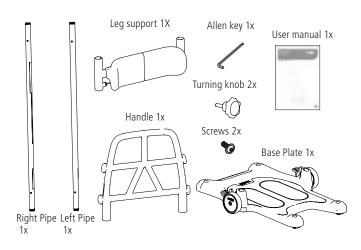

# Mounting pipes to base plate

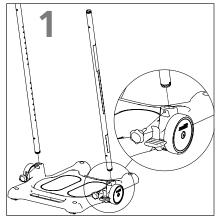

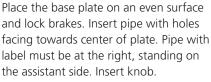

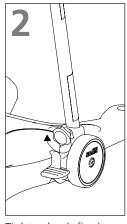


Tighten knob firmly by turning knob clockwise.



Make sure that knob are Properly fastened. Loose knobs can make the pipes wobbly and the Molift Raiser Pro unsafe to use.

# Mounting leg support

The knee cushions are adjusted individually. The leg support has 7 step adjustment from 0,3 to 0,51 meters. Follow these two steps when mounting the leg support:

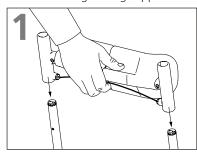


Pull release wire and slide the leg support on pipes, cushion faceing user side.




Release wire and let the leg support settle in desired position.

# Mounting handle

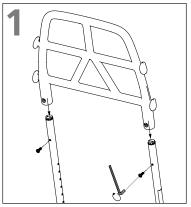

Slide the handle on pipe so holes on pipe maching holes on handle. Innsert unbraco screws and tighten with allen key.

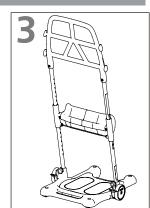

Make sure unbraco screws are pointing towards assistant side.

#### Checklist after assembly





Don't start using Molift Raiser Pro before completing the checklist after assembly.


Use this checklist to verify that the Molift Raiser Pro is Properly installed and can operate correctly and safely before use. See assembly information on the previous page for details.

- ☐ Make sure that the leg support is mounted correctly.
   ☐ Make sure that both knobs between outer pipes and base plate are mounted correctly and Properly tightened
- ☐ Make sure that the handle is mounted correctly.
- $\hfill \square$  Make sure Molift Raiser Pro moves freely.
- ☐ Make sure brake works.

# Adjusting leg support height

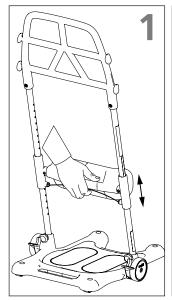

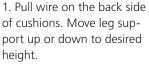

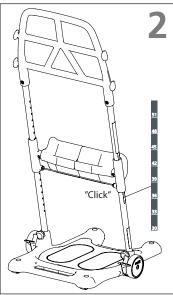


2. When correct knee support height is obtained, release wire and let the leg support settle in desired height. You will hear the leg support "click" in place and the red wire is straight. The knee support height can be read from the scale on the pipe. The number above the leg support bracket referes to the correct height.



Make sure both sides are properly settled. Push up or down until you get the auditiv feedback click.

Unattached leg support can cause knee support to slide down and make the Molift Raiser Pro unsafe to use.

## Positioning Molift Raiser Pro

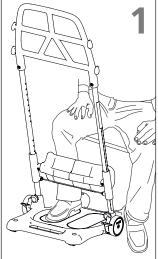




1. The user's feet are placed on the footplate.

- 2. Adjust the height of the leg support so that the top of the support is below the knee of the user.
- 3. When desired height is reached the leg support "click" in place.

# Transfering user

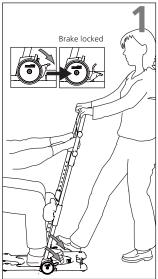

1. Lock both wheels by pressing down the brake on one side. Both wheels lock regardless of which side assistant steps on.
Place one foot on the edge

Place one foot on the edge of the base plate. Choose a grip on the handle that is comfortable for both assistant and the user.




2. Encourage the user to stand up, at the same time, counter balancing the users weight using a foot and hands (use your bodyweight by leaning backwards).



The wheels must always be locked when the user stands up and sits down.







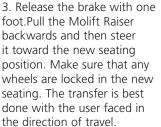



4. Lock both wheels. Place a foot on the edge of the footplate before the user sits down. Counter balance while the user sits down. Maintain eye contact with the user during the entire transfer Process.

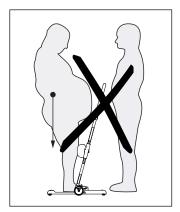

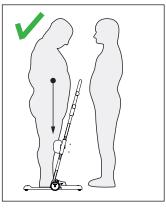

Make sure the users weight is close to the center of the base plate.

#### Accessories




Molift Raiser Soft Handels



Molift Raiser Strap



Molift Raiser Strap +

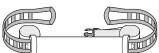

Molift Raiser Belt


# insport and storage


The carer must always counter balance the weight of the

Molift Raiser Pro can be stored and transported under temperatures between -25 - 70 °C and relative humidity between 15 - 93 % Don't use the Molift Raiser Pro for transportation of goods.

user when she/he stands up or sits down.

#### Hser / Assistant

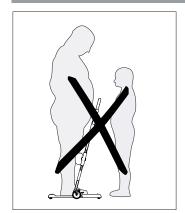

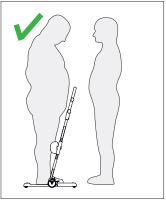

The assistant has to be able to handle the weight of the user.

# Cleaning and disinfection

Clean on a regular basis. Clean surfaces with a damp cloth using an apPropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the Molift Raiser Pro. For disinfection when needed; use isoPropyl alcohol or similar. Avoid abrasive cleaning Products. Remove hair and pile from the wheels, verify that the wheels rotates freely and brakes functions Properly. Make sure not to damage or remove labels when cleaning.

## Reconditioning

Follow cleaning and assembly Procedure. Separate reconditioning document is available at Etac.com.

#### Recycling

Molift Raiser Pro can be disposed of as metal and plastic waste. Recycling instructions is available at Etac.com

#### Spare parts

A list of spare parts is available at Etac.com.